




**Alderman Acres Mfg., Inc.**  
*Serving the Healthcare Industry since 1996*

## Wall Mounted Telescopic Curtain - Installation

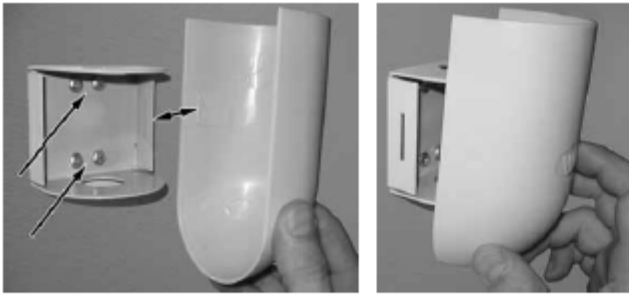

### INSTALL ROD & CURTAIN

5. Place short end of Rod Assembly through both holes on Wall Bracket; Nylon washer should make contact with top of wall bracket to give the 340 screen smooth operation when rotating.
6. "Unsnap" curtain hook latches and attach curtain hooks through holes in Vinyl Panel, then around Rod Assembly.
7. Secure hooks to Rod Assembly by "snapping" the hook latches.

### PANEL REPLACEMENT

8. To replace Vinyl Panels, remove the curtaining hooks from the vinyl panel by "unsnapping" hook latch.
9. Replace old vinyl panel with a new vinyl panel and re-install curtain hooks.

### CARE & MAINTENANCE OF YOUR SCREEN

10. PERIODICALLY CHECK ALL MOUNTING HARDWARE FOR TIGHTNESS
11. To clean Vinyl Panels, use only mild liquid soap and water.

**Wrinkles in Vinyl** – Storage and shipping causes wrinkles of the vinyl of your Winco Screen; This is normal. Please allow 1-2 weeks at room temperature (approx. 70+ degrees) to allow for vinyl to return to original appearance.

